HEAD CONTRA-ANGLE 1:1

BLUE L-40

+ GREEN L-2500

fr : Guide d'utilisation.....	03 - 09
en : User's guide.....	10 - 16
es : Manual de usuario.....	17 - 23
pt : Guia do utilizador.....	24 - 30
it : Manuale d'uso.....	31 - 37
de : Benutzerhandbuch.....	38 - 44
nl : Bedienungsanleitung.....	45 - 53

TÊTE DE CONTRE-ANGLE 1:1

fr

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HEAD CONTRA-ANGLE 1:1

en

Please read this manual carefully before operation and save it for reference in the future.

1. Description

This head is part of the BLUE L-40 3TECH and GREEN L-2500 3TECH contra-angles.
Product can be sterilized in a steam sterilizer (Autoclave), or can be used in thermal disinfectors.

Explanation of symbols

Important information for the user

CE-mark – Confirms that this product fulfills all requirements for medical products

Manufacturer

CE representative

Can be sterilized in a steam sterilizer

Serial number

Can be used in thermal disinfectors

Manufactured date

2. Technical specifications

MAX RPM	40 000 tr/min
Bur	Φ2,334~2,35 mm
Ratio	1:1
Push button	YES

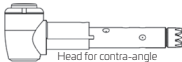
Fig. 1

3. Attaching and removing the contra-angle head

Rotate the tension ring as far as it can go as shown on Fig. 2 and hold it.

Insert the head as far as it will go, then release the tension ring and tighten it.

To remove the head, rotate the tension ring according to the direction of the arrow as far as it will go and hold it.

Then remove the head to the front and pull it out of the body.

Fig. 2

4. Insertion and removal of

Press the push-button and insert the bur into the spindle, then release the push-button (Fig. 3).

Check the proper fitting by slightly pulling at the bur.

By pressing the push-button on the back of the head, the bur can be released and then be removed.

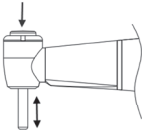
Fig. 3

5. Maintenance

Inject some maintenance oil to the hole on the intermediate gear (Fig. 4).
Attach the head to the contra angle and run it for 20 seconds.

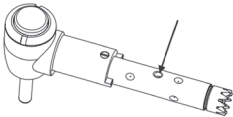
Fig. 4

Caution

Do not put this head into any chemical liquid for cleaning.

Do not clean this head with any ultrasonic cleaning machine.

6. Sterilization

This head can be sterilized in a steam sterilizer or can be used in thermal disinfectors.

This medical product has a temperature resistance of up to max. 135°C (275°F).

Only apply the following sterilization process:

Autoclave with a triple pre-vacuum (recommended): 18 minutes at 134°C.

7. Warranty

The distributor provides the end user a warranty for proper function, immaculate material and workmanship for a period of 1 year.

In case of justified complaints the distributor carries out a free repair or a possible free replacement. This will be decided by the distributor.

The distributor is not liable for defects and their consequences that have arisen whose arising may be due to natural wear, improper handling, improper cleaning, or maintenance, non-compliance with operating or manual instructions, calcination or corrosion, contaminated air and water supply or chemical or electrical influences that are unusual or not permitted according to the distributor's instruction for use or to other manufacturer's instructions.

All liability is excluded if defects or their consequences arise because of interventions or modifications on the product by the end user or by a third party not authorized by the distributor.

Warranty claims will only be accepted if the product is submitted with a proof of purchase in the form of an invoice or a copy of the delivery note.

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Being Foshan Medical Equipment Co., Ltd.
Shanghai International Holding Corp. GmbH