**Wellkang Ltd.**

Suite B, 29 Harley Street, London W1G9QR, UNITED KINGDOM

**Foshan COXO medical instrument Co.,Ltd.**

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### 1. Safety caution

#### WARNING

Please read the instruction manual thoroughly before using this equipment, this will guide you to operate it properly.

This device must be used in strict accordance with the instructions contained in this user's guide. The manufacturer will not take any responsibility for damage caused by improper use.

- Before connecting the base of the charger or the handle to the main power, make sure that the power voltage is in accordance with the charger voltage range. Improper voltage could damage the device and cause the danger to the patient or the operator.
- Don't look at the light with your eyes without protection, it's harmful to your eyes. In order to avoid the eyes viewing the light directly, please assemble the eye protector.
- Before using, let the device reach the room temperature. Especially when the device has been moved from a cold place to a warm place.
- Never use any other battery except the original battery; otherwise it could result in severe damage.
- Don't put the device near flammable items and don't touch flammable liquid with this device.
- To avoid electric shock, don't insert any other parts into the device; it may cause electric shock or damage.
- Please use our accessories, such as: light guide rod, adaptor, and battery. We won't take the responsibility for any problem or damage because of using other parts which are not supplied by us.
- Avoid getting detergent into the device, in case it causes short circuit or damage.
- It is not allowed to modify the device for any reason. Any disassembling or modification will cause the invalidation of the guarantee.
- Unstable voltage and if it's in electromagnetic environment will interfere the normal operation.
- Do not use the device with patients who have heart pacemakers.
- When discarding the accessories, please abide with the local regulation, if necessary, contact our service center to take it back.

### Standard configuration

Caries diagnostic handle	1
Charging base	1
Light guide rod (90°)	1
Light guide rod (120°)	1
Adaptor	1
Goggles	1

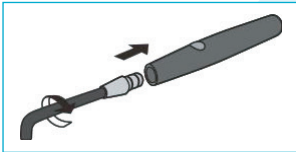
### 2. Intended use

This product is used for dental surgery, to identify particular fluorescence properties of bacteria on the caries, it makes it easier to target and remove the infected substance on the tooth. This product is only intended for dentists who have already been trained and for use only in clinics and laboratories.

### 3. Construction, installation and use


This product is consisted of a light guide rod, caries diagnostic handle, charging base, adaptor and goggles.

## Install the light guide rod

Insert the light guide rod straight into the handle until you hear a slight clicking sound and feel that it clicks into place.

## Use

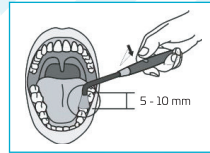
1. Please put on the goggles before using, to prevent the eyes from being damaged by the blue light.
2. During the treatment, put the end of the optic fiber to the tooth (the distance between the optic fiber and the tooth should be about 5-10mm).
3. When the equipment is on standby, press the power button to turn on, the optic fiber will output light, pressing the button again will shut off.
4. Teeth will be displayed fluorescent when the light shines on them, the fluorescent results of the light probe provide information about the cavity's surface, which can aid treatment. The fluorescent green areas indicate healthy tooth substance; the fluorescent red areas show the caries.

## Battery

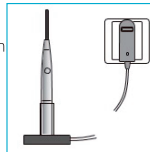The first time you use the equipment, please make sure it is fully charged, this process takes about an hour and a half.

## Charge

1. When the unit is in use and at low voltage, light will flash every 1s to remind user that it should be charged.
2. Please connect the corresponded power source, the indicator of the adaptor will turn on. Then put the unit into the charging base.
3. Put the unit into the connected charging base, the charger indicator will turn red indicating the battery is charging.
4. After finishing charging, the indicator in the charging base will turn green, the unit will automatically stop charging.

## 4. Routine maintenance and cleaning

- We suggest the unit should be cleaned and sterilized before the first using.
- Avoid using detergent or materials which contain flammable ingredients, if you do, make sure it completely evaporates before starting operation.
- When cleaning the handle and the battery, the handle and the base must be separated completely.

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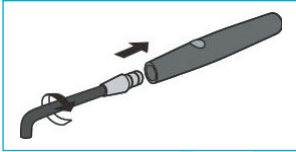
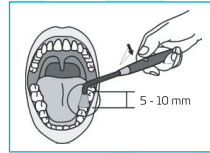
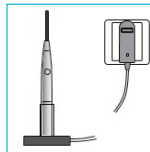
When it is used for next patient, the light guide rod must be cleaned and sterilized.

- The main body and the accessories can be cleaned with alcohol or neutral detergent, do not use caustic or rubbing detergent, and no ultrasonic cleaning.
- Don't make the light guide rod touch any hard items after using in case of scratching the surface, please check if there is any material on the terminal, if yes, clean it with plastic tool, don't use metal instrument. If light guide rod is damaged, it is better to replace it with a new one.
- The light guide rod can be cleaned with alcohol, or can be sterilized for 18 minutes at 134°C.
- Please put the handle on the charger base if stopping working for a short time, but for extended periods when not in use, please take out the light guide rod and battery, save them in a good condition.

### 5. Storage and transportation

- This equipment should be moved carefully avoiding any shocks, and installed or kept in a place that is dry and cool.

### 6. Warranty

The warranty of this equipment is 12 months, battery and adaptor are 6 months, other accessories are not included in the warranty.

### 7. Technical specifications

Power adaptor	Input: 100-240Vac 50/60Hz Output: DC 5V/ 0.5A
Li- ion battery	Voltage: DC 3.7V Capacity: 320 mAh
Temperatures Operating	+10°~+40°C
Temperatures Storage	-10°~+55°C
Humidity Operating	30~75% RH
Humidity Storage	20~85% RH
Atmospheric pressure Operating	80~106 kPa
Atmospheric pressure Storage	50~106 kPa
Protection against electrical shock	Type B

## 8. Symbols and definition

	Class II equipment
	Direct current
	Type B equipment
	CE Marking
	Use indoor only
	WEEE
	Please, refer to the instructions
	Atmospheric pressure
	Sterilizable in an autoclave up to maximum 135°C

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