HEATER

TC-70

Shanghai International Holding Corp. GmbH
Eiestrasse 80, 20537 Hamburg, GERMANY

 Foshan Dade Medical Technologie Co., Ltd
F4-4. Block 2, No 117 Zhangcha 1 Road,
Chancheng Foshan, Guangdong, CHINA

CADENCE
2 bis, chemin du loup
93290 Tremblay-en-France - FRANCE

Importateur – Importer – Importadora – Importador – Importatore – Importeur – Invoerder

Réchauffeur pour composites et carpules d'anesthésie
Heater for composites and anaesthesia cartridges
Calentador para composites y cartuchos de anestésicos
Aquecedor para compósitos e carpules de anestesia
Riscaldatore per compositi e carPELLI anestetici
Heizgerät für Verbundwerkstoffe und Anästhesiekarpulen
Verwarmer voor composieten en anesthesiecarpels

HEATER TC-70

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en

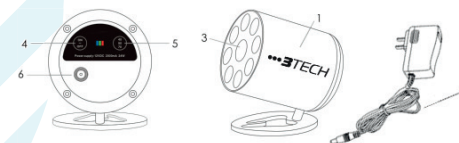
1. Product Description

HEATER TC-70 is designed to heat composite resin materials to improve fluidity performance and adaptation to dental cavities. It can also be used to heat the anaesthesia carpels to inject the anaesthetic product without pain.

HEATER TC-70 uses a precision circuit system to control its heating module, maintaining a temperature tolerance of ±1°C. Depending on the material and your personal choice, the temperature can be set at 40°C, 50°C or 70°C.

2. HEATER TC-70 composition

1. Heating body
2. Adapter
3. Temperature indicator
4. On/Off button
5. Temperature selection button
6. Power DC jack

3. Operating

1. Insert the DC plug of the power adapter into the DC socket of the heating body. Then plug the power adapter into an electrical outlet. At this time, three temperature indicators are on at the same time (standby heating).
2. Place the heated body on a flat table.
3. Press the button on/off to turn on the heated body. Press the button again to turn off the heating body.
The default temperature for heating the main body is 40°C.
When the heating body is off, three temperature indicators are on at the same time.
4. Press the temperature selection button to select the temperature you need.

Please consult the detailed temperature list:

Temperature	Indicator light color	Heating	Reach preset temperature
40°C	Blue	Blue LED shining	Blue LED is always on
50°C	Green	Green LED shining	Green LED is always on
70°C	Red	Red LED shining	Red LED is always on

- 40°C for composites and anaesthesia cartridges
- 50°C for composites
- 70°C for composites

When indicator light changes from blinking to steady light that indicates that the temperature has reached the preset temperature.

5. Insert the composite resin (syringe or capsule) or anesthetic carpule into the hole in the heating body.

NOTE

Insert the composite resin with its cap to prevent the material from leaking inside the heating body.

4. Cleaning and disinfection

All parts can be wiped clean with alcohol-free wipes.
Do not immerse any parts in any liquid.
Cannot be autoclaved.

5. Technical parameters

- Power: 24W
 - Temperature setting 40±1°C, 50±1°C and 70±1°C
- Power Adapter
- Enter: 100-240V AC 50/60Hz 0.8A Max
 - Output: 12V DC 2A

6. Warranty

All parts of this equipment are warranted for 1 year. Man-made damages are not covered by the warranty. Return the unit to your distributor for repair in the event of a problem. Do not disassemble the equipment for maintenance. If you disassemble and repair it yourself, you will no longer be covered during the warranty period.

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