MedNet EC-Rep GmbH
Borkstrasse 10, 48163 Muenster, GERMANY

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CONTRA-ANGLE IMPLANT POWER PRO

Please carefully read instruction manual before first

1. Product introduction

The contra-angle handpiece is driven by a micro-motor of implanting device through driving the movement to rotate at a specified speed, so as to drive needles / drills which has specific functions. It is applicable to drilling, tapping, and implanting in dental implant surgery.

Features:

- Smooth drive; low noise.
- Gear carrying capacity is big enough to insert and remove the implants.
- Long service life
- Adaptable to all types of implanting device.

en

1.1 Technical data

Rotate speed	15~2000 rpm
Chuck	Push button chuck
Reduction gear ratio	20:1
Nozzle type	External / Internal
Needle / drill type	Ø 2.35 Surgical needle / drill (ISO1791-1)
Needle / drill length	11.6 mm
Maximum length of needle	36 mm
Maximum diameter	Ø 4.7 mm
Type of bur	With plane and slot (Contra-angle connection)
Size of bur	ISO1797-1 Ø 2.35 mm

Minimum length of bur	22.5 mm
Light source	LED
Light	With light

1.2 Product performance

This product can be used to drill, tap and implant in dental implant surgery by using an implanting system with a maximum rotating speed of up to 40,000 rpm.

1.3 Structure compositions

The product consists of head, driving shaft and body.

1.4 Packing list

Contra-angle handpiece	Cleaning needle
Lubricating fitting	Center nozzle

Nozzle holder	Instruction Manual
Y shape water hose	

1.5 Scope of application

It is applicable to drilling, tapping, and implanting in dental implant surgery.

1.6 Operation conditions

- Environment temperature: +5°C~+ 40°C
- Relative humidity: 30%~75%
- Atmospheric pressure: 70 kPa~106 kPa

1.7 Warnings

- Please check the operation outside oral cavity before treatment. Check carefully whether there is vibration, noise, or heating. If there is any unexpected problem, please stop using.
- Avoid strong impact and dropping.

- Do not arbitrarily disassemble or modify the handpiece.
- Please prepare a backup handpiece to cope with the case that handpiece malfunction during operation.
- Do not use non-recommended needles / drills. Otherwise, the running needle / drill may bend or fly out.
- Please keep the needle handle / drill handle clean. Otherwise, the stains inside the chuck device will affect the correct fixing of needle, causing the needle to fly out during operation.
- Do not use the contra-angle without needle/dental drill.
- Do not press the button while operating. Otherwise, it may cause head overheating or the needle flying.
- Cooling water should be supplied during operation. Otherwise it may cause overheating.
- For safety, please wear protective goggles or dust mask while cutting.
- Do not exert too much force to treatment site.
- Please carry out regular maintenance and inspection.
- If the contra-angle has not been used for a period of time, please check whether there is noise, vibration, or overheating while it is running before operation.
- Damage or serious wear will lead to increased transmission resistance, abnormal sound and so on. As a result, the planting efficiency is reduced; the drill will shake greatly; and there will be abnormal sound, etc. The operator should change the contra-angle regular according to clinical conditions. If the contra-angle wears to a certain extent, please change a new one.

1.8 Contraindications and warnings

- The hemophilia patient is forbidden to use this equipment.
- The patients or dentists with heart pacemaker are forbidden to use this equipment.
- Heart disease patients and children should be cautious to use the equipment.
- Patients with oral and maxillofacial infections, various types of oral mucosal diseases that have not been cured, periapical disease, gingivitis, periodontal disease and oral cancer are forbidden to use this equipment.
- Patients with allergic constitution and a history of drug allergy are forbidden to use this equipment.
- People with mental disorders should be cautious to use this equipment.
- Patients with severe systemic infection or systemic diseases such as diseases in heart, liver, kidney, hematopoietic system, digestive system and endocrine system should be cautious to use this equipment.
- Pregnant women, lactating women, or women of childbearing age who have a plan of pregnancy should be cautious to use this equipment.

2. Instructions

2.1 Water injection nozzle

CONTRA-ANGLE IMPLANT POWER PRO offers three types of water injection modes: external water injection, internal water injection, and combination of both the former injection modes. It depends on the parts and operating procedures.

2.1.1 External water injection

Connect the water injection hose to the external nozzle firmly. (Figure 1)

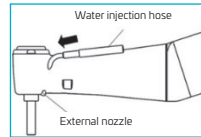
Fig. 1

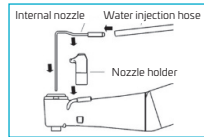
Fig. 2

2.1.2 Internal water injection

Only internal water injection dental drill can be used.

- Place the nozzle holder steady.
- Plug the water injection hose into internal nozzle.
- Plug the internal nozzle into the head.
- Place the internal nozzle into the nozzle holder. (Figure 2)

2.1.3 Internally and externally inject water at the same time only

Only internal water injection dental drill can be used.

- Connect the Y shape water hose to the external and internal nozzles.
- Connect water injection hose to Y shape hose. (Figure 3)

2.2 Installation/disassembly of handpiece

2.2.1 Installation

Align the contra-angle handpiece with the motor plug, and then insert the contra-angle handpiece into the motor and tighten it. If the contra-angle handpiece is properly locked, there will be a "click" sound.

2.2.2 Disassembly

Hold the motor and contra-angle, and then pull the contra-angle out. (Figure 4)

⚠ NOTICE

- Install and disassemble the contra-angle handpiece only after the motor is completely out of operation.
- Connect only to the adapter motor with a maximum speed of less than 40,000 rpm.
- The adapter of this product is in accordance with ISO3964 standard.

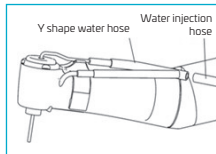
Fig. 3

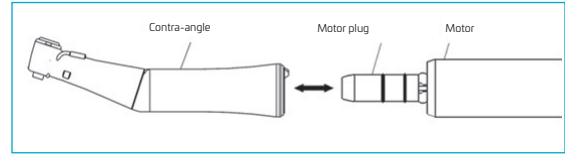
Fig. 4

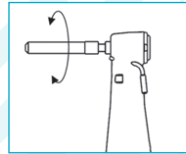
Fig. 5

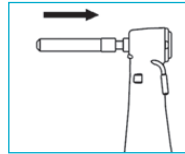
Fig. 6

2.3 Installation/Disassembly of needle/dental drill

2.3.1 Installation

- Plug the needle/dental drill into the head.
- Gently turn the needle / dental drill to trigger locking device. (Figure 5)
- Push the needle / drill inward. If the needle / drill are tightly locked, there will be a "click" sound. (Figure 6)

⚠ NOTICE

- After the needle/drill is locked, gently pull the needle/drill to ensure that the needle/drill is securely locked.
- If the needle/drills are not installed securely, they may fly out or be hard to be removed.

2.3.2 Disassembly

Press the button with your thumb and remove the needle / dental drill. (Figure 7)

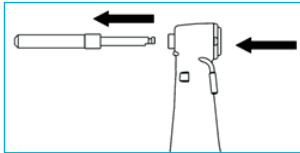
Fig. 7

NOTICE

Press the rear cover button to flush with the head cap surface.

3. Cleaning, disinfection and sterilization

3.1 Cleaning of contra-angle

3.1.1 Removal

- Remove the contra-angle from motor.
- Remove the nozzle holder, internal nozzle and burs/bits from the contra-angle.
- Unscrew the screw on head with hex wrench. (Figure 8)
- Unplug the head from the body. (Figure 9)
- Unplug the driving shaft from the body. (Figure 9)

3.1.2 Cleaning

- Thoroughly clean the removed parts with a neutral or mild alkaline detergent.
- Rinse the cleaning solution thoroughly with tap water.
- Immediately install the contra-angle after flushing, wipe the surface to clean, and then lubricate it with oil. Oiling can drain the internal water inside handpiece.

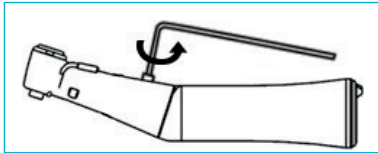
Fig. 8

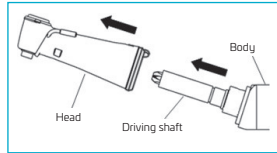
Fig. 9

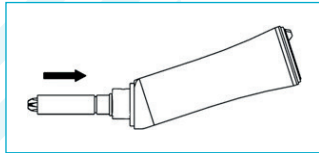
Fig. 10

Fig. 11

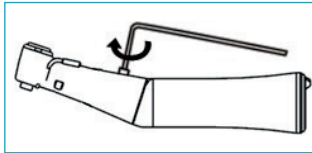
Fig. 12

⚠ NOTICE

Do not wipe the removed parts with a brush. The hair on top of the brush will attach to the part and enter the interior of the product to cause product failure.

3.1.3 Installation

⚠ NOTICE

When disassembling and installing, follow the order of assembly, otherwise it may cause noise or product performance defects.

- Press button on the back cover of head till it is flush with the cap surface of head.
- Plug in the driving shaft. (Figure 10)
- Rotate the gear at the end of the drive shaft to engage. (Figure 11)
- Align the mark on the head with the mark on the body. Insert the head and body directly. Make sure that there is no gap between the head and the body. If there is a gap, the head and body gears will fail to match well. At this time, remove the head, rotate the gear at the end of the drive shaft, and insert the head again.
- Rotate the screw into the head. (Figure 12)

3.2 Cleaning of nozzle

- Use a cleaning needle to clean the internal and external nozzles. (Figure 13, Figure 14)
- Connect all the nozzles to water hose, and then rinse with water.

3.3 Disinfection

- After the cleaning of contra-angle handpiece, wipe the surface of contra-angle handpiece with a soft, clean cloth moistened with methanol, ethanol or phenolic alcohol.
- Do not use oxidizing potential water (strong acid, super acid) or disinfectant to clean, soak or wipe the contra-angle handpiece.
- Do not dip the contra-angle handpiece into the disinfectant.

NOTICE

It's not recommended to clean with ultrasonic.

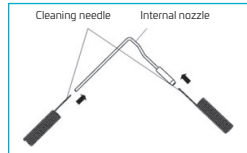
Fig. 13

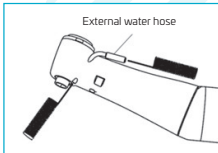
Fig. 14

3.4 Sterilization

After each treatment, please sterilize under high temperature and high pressure as per the following steps.

- Clean the contra-angle handpiece with method set out in 3.1.2.
- Inject lubricant into contra-angle handpiece.
- Place the contra-angle handpiece into a sterile bag and seal it (only when it is autoclaved under vacuum condition).
- Steam sterilization at 134°C, 2.0 bar-2.3 bar (0.20 MPa-0.23 MPa), for 18 minutes.

WARNING:

- Use only autoclaves that are matching the requirements of EN 13060, EN 285.
- Use a validated sterilization procedure according to ISO 17665.
- Respect the maintenance procedure of the autoclave device given by the manufacturer.
- Use only this recommended sterilization procedure.
- Control the efficiency (packaging integrity, no humidity, color change of sterilization indicators, physico-chemical integrators, and digital records of cycle's parameters).
- Maintain traceability of procedure records.
- To keep it clean, the contra-angle handpiece that has been sterilized should be placed in a bag that has been sterilized under high temperature and high pressure.

⚠ NOTICE

- If there are residual chemicals on the device, please do not autoclave the device. Otherwise it may lead to surface coating loss or damage to the internal components.
- Keep the contra-angle handpiece in a place that is free from influence of dust, sulfide or salt in the air.
- Do not touch the contra-angle handpiece immediately after sterilization under high temperature and high pressure, because the contra-angle handpiece is very hot.

4. Maintenance

4.1 Lubrication

- Use Q-type nozzle to inject oil.
- Inject oil after removing the needle.
- Shake the lubricant spray can to mix lubricant well with the propellant.
- Insert the nozzle into the end of the contra-angle handpiece. Put a tissue on the head of the contra-angle handpiece and keep the contra-angle handpiece spraying for 2-3 seconds. Check whether the tissue is clean. If it is clean, the contra-angle handpiece can be autoclaved. If not, spray again until the tissue is clean. (Figure 15)

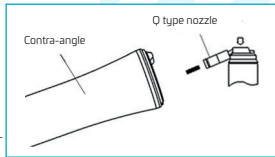
Fig. 15

⚠ NOTICE

- Hold the contra-angle handpiece tightly to prevent the contra-angle handpiece from slipping as a result of jet pressure during oiling.
- Oiling should be repeated until the contra-angle handpiece is clean and free from stains. If the blood remains on the contra-angle handpiece, blood clotting may cause corrosion, resulting in phenomena such as low rotation speed or torque.
- Keep the lubricant spray can upright.

5. Storage and transport conditions

- Environment temperature: $-20^{\circ}\text{C}\sim+55^{\circ}\text{C}$
- Relative humidity: 10%~93%
- Atmospheric pressure: 70 kPa~106 kPa

This product should be placed or stored in a dry and clean environment which is away from acid, alkali and other harmful chemicals and gases.

6. Environment protection

Please dispose according to the local laws.

7. Warranty

The manufacturer grants the user a 12 months guarantee for its complete product range, from the date of invoice issued. Maintenance over the term of guarantee will be at the customer's charge.

The manufacturer will not be responsible for damage or injury resulting from:

- excessive use
- improper manipulation of the product, or modification to product carried out by persons not authorized by the manufacturer.
- fail to follow the instruction to install, operate and maintain the contra-angle.
- damage chemical, electrical or from electrolysis due to improper autoclaving and storing.
- improper working pressure.

The technical specifications and dimension in the instruction are given only as a guide. They may not be subject of any claim..

8. Symbol description

IPX0 General equipment

Can be autoclaved

Consult the accompanying documents

Date of manufacture

Follow Instructions for Use

Products comply with WEEE directive

Atmospheric pressure for operation and storage: 70 kPa–106 kPa

Used indoor only

Manufacturer

Serial number

Humidity limit for storage: 10% ~ 93%

Temperature limit for storage: -20°C ~ +55°C

9. Statement

The manufacturer reserves the right to change the design of the equipment, the technique, fittings, instruction manual and the content of the original packing list at any time without further notice. The Figures are only for reference. The final interpretation rights belong to the manufacturer. Please refer to product packaging label for date of manufacture.

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