...3TECH

LED CURING LIGHT LED-1007

LAMPE LED À PHOTOPOLYMERISER SANS FIL

WIRELESS LED LIGHT CURING UNIT

LÁMPARA LED SIN CABLE

FOTOPOLIMERIZADOR LED SEM FIOS

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Introduction

Thank you for choosing **Curing Light LED-1007**. For safety reasons and to achieve the best results, please read the user's guide carefully before using the product.

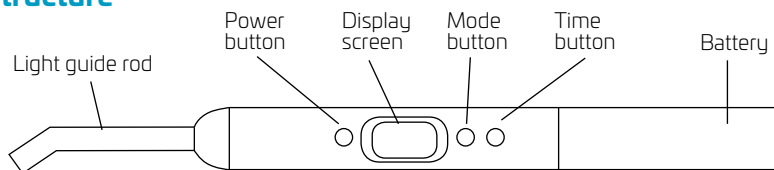
This user guide describes the normal properties of the Curing Light LED-1007, and introduces the principles of the safety issues, installation, usage and maintenance.

We are always trying to improve our products, this may result in small changes of the components.

Before the first use, please check the accessories and make sure they are all in good condition.

1. Main unit, 2. Light guide, 3. Battery, 4. Charge, 5. Adapter, 6. Light cover plate,
7. User manual


Structure

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Size dia 22 x 175mm (LED Light)
Weight 165 gr
Safety class Ordinary type B, non-continuous service, IP 40, Class II
Input voltage 100-240 V, 50-60 HZ
Battery capacity 2200mAH
Wavelength 430-485nm
Intensity 5W>1100 mW/cm ²
Polymerisation 10s, > 3mm (5W)

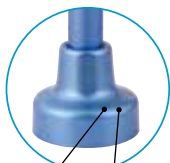Usage


Yellow Green

Battery charge

Connect the adapter to the AC power and connect then adaptor to charger. Place the handpiece on the charger. When charging, the charging indicator (**yellow**) is on and the power indicator (**green**) is off. When the battery is fully charged, the charging indicator (yellow) is off and the power indicator (green) is on.

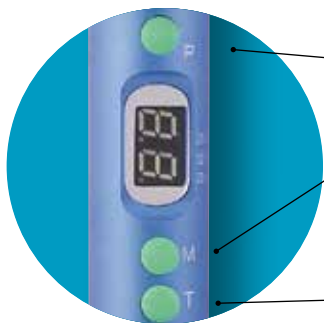When using a new battery for the first time, it is best to charge it for 6-7 hours, but it should not be charged for over 7 hours. When battery is full charged, please disconnect the AC power.

Operation

- A) Power on/off:** When the unit is off or standby, press power button to start the unit working, the light output shows a blue light, press the button again to stop working and turn to standby.
- B) Mode select:** Press mode button to select three types of work mode:
01 Gradual/Ramping mode
02 Flashing mode
03 Full light mode
- C) Time select:** Press Time button to select time. Four time settings: 10/20/30/40 seconds can be chosen.
- D) When the unit is working,** the buzzer will beep every 5 seconds.
- E) Automatically power down:** When the unit is in standby without the button being pressed for 8 minutes it will turn off automatically.
- F) Low battery alert:** If when starting work the unit power and the voltage of the battery is lower than the required level the unit will give an alarm; the buzzer beeps 5 times and the indicator flashes the stage. Please charge the battery in time.

Notice

1. The working time between each handling is not less than 30 seconds.
2. Please replace the light guide when it has cracked or show scratches.
3. Disinfection of the light guide: immerse the rod in 75% alcohol several minutes.
4. When the battery is fully charged, please disconnect the AC power.
5. Avoid touching the electrodes of the battery with metal.
6. Please use a swab with anhydrous alcohol to clean the pins in the charger regularly, this should avoid weak contact from charger to handpiece.

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FOSHAN CICADA DENTAL INSTRUMENT CO., LTD
Lotus NL B.V.