# VIBRAMIX

## VB-4000

EC REP

**Lotus NL B.V.**

Koningin Julianaplein 10, 1e Verd, 2595AA, The Hague, Netherlands.  
E-mail: peter@lotusnl.com

Amalgamators

CE

**Foshan COXO medical instrument Co.,Ltd**

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en

## ⚠ NOTICE

PLEASE REMOVE THE TWO SCREWS AT THE VIBRAMIX VB-4000 BOTTOM BEFORE USE!

## 1. Introduction

Please read this manual carefully before you begin to use VIBRAMIX VB-4000 and store it close to the unit. Thank you for purchasing VIBRAMIX VB-4000. VIBRAMIX VB-4000 is simple to operate and uses a microprocessor to control the unit; an exact running time can be set on it.

## 2. Usage

This mixer is used to mix dental capsules, for example silver with mercury in capsule form to make alloy in the best manner. Unlike the manual method, VIBRAMIX VB-4000 makes sure the compound is mixed to optimum consistency and decreases the risk of mercury pollution in the dental office.

## 3. Technical data

Electric safety	Class II, type B
Power supply	Europe: AC 220 ± 22 V, 50/60 Hz or UK: AC 110 ± 11 V, 60 Hz
Input power	200 VA
Oscillation frequency	> 4000/Min
Fuse	Europe: 2 A (220 V), Ø 5×20 mm or UK: 3 A (110 V), Ø 5×20 mm
Use environment	Temperature: 5 °C ~ 40 °C, relative humidity: ≤80 %
Store environment	Temperature: -40 °C ~ +55°C, relative humidity: ≤80 %

## 4. Operating method

Open the transparent protective cover, fix the capsule on the holder of VIBRAMIX VB-4000, then close the cover.

### Time setting

- ① Press the ↑ key to set the timer to run for longer, press the ↓ key to set time to less.
- ② Press the ▶/■ start button, the mixer will begin its cycle, and will stop after the specified time. To stop the unit at any point during the cycle, press the ▶/■ start button once again.
- ③ When the mixer has finished, you can open the cover and remove the capsule.

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## 5. Avertissements

Use protect ground seat plug.  
Before starting the unit, the protective cover should always be closed.

## 6. Notice

The rotating parts in the VIBRAMIX VB-4000 must be lubricated periodically.

### ATTENTION

Before use, remove the two transport screws which are used to protect the unit during transportation.

## 7. Maintenance

N°	Trouble	Solution
1	The LCD has no numeric display	Check fuse
2	The capsule holder swings powerlessly	Check the driving rubber hand

## 8. Garantie

The unit has a 12 months guarantee. Depending on the fault, the unit will be replaced, repaired or parts supplied. Disposable parts such as the fuses and capsule holder are not included.

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