# SCALER HANDPIECE

L-28

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TYPE S

TYPE E

...3TECH

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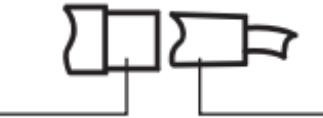
# SCALER HANDPIECE L-28

Please carefully read this Instruction for Use before use

## 1. Description of device

### 1.1 Description

The SCALER HANDPIECE L-28 is the accessory of scalers. It is the handheld energy transformation part of the Ultrasonic Piezo scaler, by using piezoceramics to transfer electric energy to mechanical energy to eliminate the dental calculus.

### 1.2 Product structure

The SCALER HANDPIECE L-28 is composed of a cover and an ultrasonic transducer.

### 1.3 Scope of application

The SCALER HANDPIECE L-28 is used for dental calculus elimination

14

and root canal cleaning.

### 1.4 Contraindication

- Hemophilia disease patients are not allowed to be treated with this equipment.
- Patients with a heart pacemaker are forbidden to be treated with this equipment.
- Doctors with a heart pacemaker are forbidden to use this equipment.
- Heart disease patients, pregnant women and children should be treated cautiously with this equipment.

### 1.5 The classification of the device

- Operating mode: Continuous operation.
- Protection degree against water (used on the pedal): IPX1.
- Degree of safety of application in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide: Equipment not suitable for being used in the presence of a flammable anesthetic mixture with air, with oxygen or nitrous oxide.

## 1.6 The main technical specifications

- Power source Input: ~190 V – 230 V
- Output primary tip vibration excursion: < 90 µm
- Output tip vibration frequency: 28 kHz ± 3 kHz
- Output half-excursion force: 0.5 N ~ 2 N
- Tip's output power: 3W ~ 20W
- Water pressure: 0.01 MPa ~ 0.5 MPa

## 1.7 Working conditions

- Environmental temperature: +5 °C ~ +40 °C
- Relative humidity: 30 % ~ 75 %
- Atmospheric pressure: 70 kPa ~ 106 kPa

## 2. Installation of the product

### 2.1 Installation of handpiece

- Open the packing box, make sure that all the parts and accessories are

complete according to the packing list. Contact the dealer if necessary.

- Keep the connection between the detachable handpiece and the cable dry before installing the handpiece.
- Make sure that the handpiece is fixed to the cable correctly. (**Picture 1**).

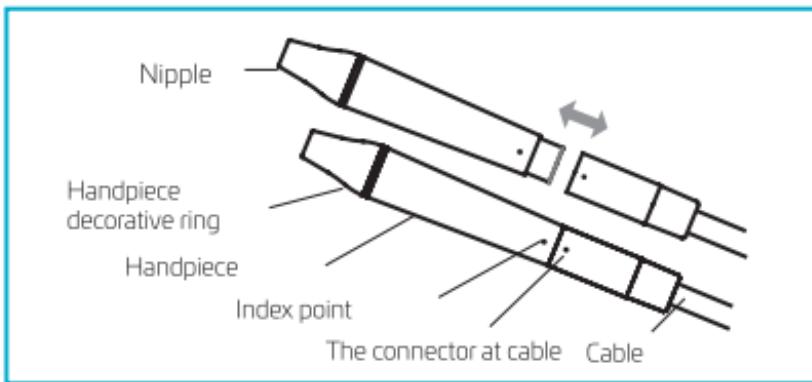
Figure 1

### 2.2 Installation of tips

- Fix the tip to handpiece with the torque wrench. (**Picture 2**).

- Please use compatible scaling tips

**Notes :** The detachable handpiece is a high-tech device which should be used carefully. Don't use too much torque force to screw the tip on, screw the tips on with the torque wrench. In order to prevent accidental vibration by touching the foot switch, unscrew the scaling tips after each operation.


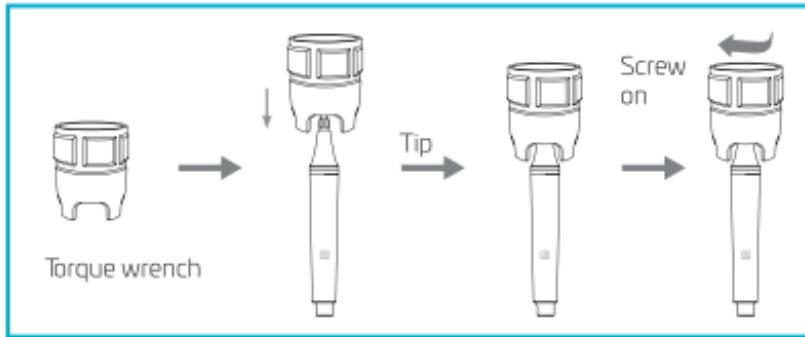
Figure 2
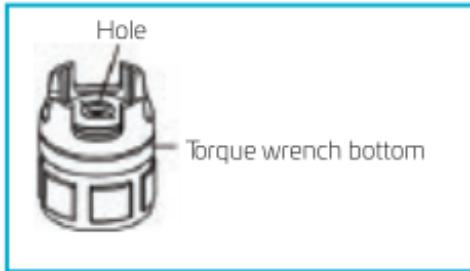
### 3. Product function and operation

#### 3.1 Scaling function

Connect the handpiece and the connector of cable of the main unit of the scaler correctly, and adjust the water and vibration intensity to a suitable level.

- The handpiece can be handled the same way a pen is handled.
- Select a suitable scaling tip as you need, screw it on the handpiece tightly with the torque wrench.
- During the clinical treatment, be sure not to make the end of tip touch the teeth vertically.

- Do not overexert the tip on the surface of the teeth in case of hurting the teeth and damaging the tip.
- After finishing operating, keep the machine working for 30 seconds on the water supply condition in order to clean the handpiece and the scaling tip.

## 3.2 Endo function

### Usage process

- Fix the endochuck to the handpiece with the torque wrench.
- Unscrew the screw cap of the endochuck.
- Put the ultrasonic file into the hole of the endochuck.
- Screw down the screw cap with the endo wrench to tight up the ultrasonic file.
- Press option key, turn to endo root canal cleaning function.
- Press option key, turn the water and vibration intensity to a suitable level.
- Put the ultrasonic file into the patient's root canal slowly, step on the foot pedal, then make endo root canal cleaning. During the treatment, turn up the power gradually according to your needs.

### Notes

- When fixing the endochuck, it must be screwed down.
- The screw on the endochuck must be screwed down.
- Don't apply to much pressure when inserting the ultrasonic file into the root canal.
- Don't start up the foot pedal until the ultrasonic file is in the root canal.
- The power range is supposed from the 1<sup>st</sup> to the 5<sup>th</sup> level.
- Please sterilize the endochuck and handpiece after operating.

## 3.3 Instruction for main components of detachable handpiece

- Nipple** : the nipple can be removed for easy cleaning.
- Light pipe**: light guide, can be autoclaved under high temperature and pressure.
- LED lamp**: illuminates the oral area, detachable and sterilized.
- Handpiece**: the main part of the whole handpiece can be autoclaved under high temperature and pressure.


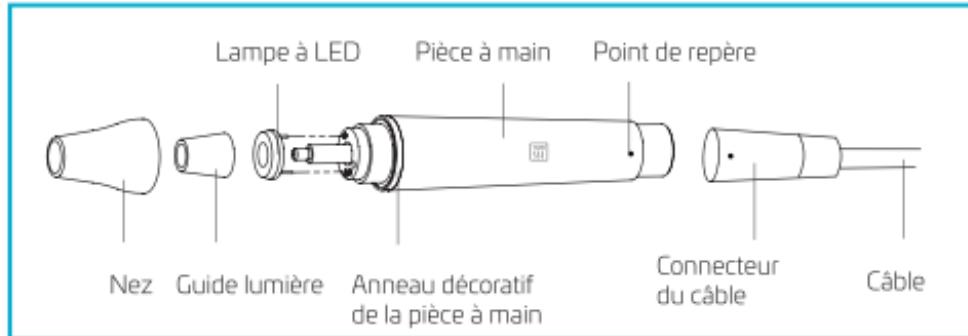
Figure 3
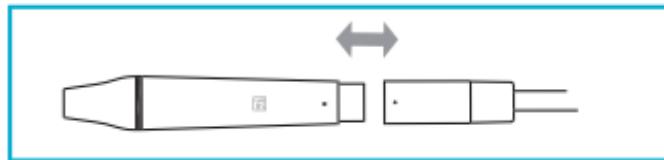
## 4. Notes

### ⚠ ATTENTION

- Don't knock or rub the handpiece.
- Don't screw on or unscrew the scaling tip and the endochuck while stepping on the foot pedal.
- The handpiece, scaling tip, torque wrench, endo wrench and endochuck must be sterilized before each treatment.
- The internal screw threat of the scaling tips produced by some manufacturers can be coarse, rusty or collapsed. This will damage the external screw threat of the handpiece irretrievably. Please use the manufacturer's brand scaling tips.
- *We are only responsible for the safety on the following conditions:  
The maintenance, repair and modification are made by the manufacturer or the authorized dealer.*

## 5. Trouble shooting

Fault	Possible causes	Solutions
Handpiece doesn't work	The tip is loose	Tighten it
	Something wrong with the handpiece	Take out the handpiece and send it back for repair
	Something wrong with the cable	Contact the distributor
	The connect plug between the cable and circuit board is loosen	Contact the distributor
There is no spray when getting through the electricity	The water control switch is not "on"	Turn on the water control switch
	No pressure or low pressure	Check the pressure
There is no water when getting through the electricity	The water control switch is not "on"	Turn on the water control switch
	There is impurity in the electric- magnetic valve	Contact our dealers or us
	Water way clogging	Clean the water line by three-way syringe

Fault	Possible causes	Solutions
Low vibration	The tip hasn't been screwed on the handpiece tightly	Screw the tips tightly by torque wrench
	The tip is shaken loose by vibration	Screw on the tip tightly
	The coupling between the handpiece and the cable isn't dry	Dry it, especially the water between handpiece and connector
	The tip is worn out or deformation	Change another tip
The handpiece generates heat	The water control switch is in a low setting	Turn the water control switch to a higher grade
LED light don't work	Poor contact	Contact tightly
	Something wrong with LED light	Change a new one
	LED lamp installed backwards	Please install the "+" of the LED lamp to the "+" of the handpiece
There is water seeping from the coupling between the handpiece and the cable	The waterproof "O" ring is damaged	Change a new "O" ring

If the problem still can't be solved, please contact the local dealer.

## 6. Cleaning and sterilization

- Clean and sterilize the handpiece after using it.
- Remove the nipple, light pipe, and LED lamp before autoclaved, clean by pure water and dry it by three-way syringe. (*Picture 3*)
- Autoclave at 134 °C while 18 minutes.

Using sterile towels or sterilized bags pack the handpiece, nipple, light pipe and LED lamp before autoclaved, install them as shown in picture 3 after cooling down.

### Notice

- Clear the cleaning liquid in the handpiece with three-way syringe before sterilization.
- Be sure that the scaling tip has been unscrewed from the handpiece as it cannot be sterilized with other pieces.
- Please notice whether the exterior of the handpiece is damaged during the treatment or sterilization.
- Please don't smear any protective oil on the surface of handpiece.
- Please take out the handpiece, nipple, light pipe, LED lamp as Picture 3

and sterilize separately.

- There are two waterproof "O" rings at the end of the handpiece. Please lubricate them with dental lube frequently, as sterilization and repeated pulling and inserting will reduce their using life. Exchange it for a new one once it is damaged or worn excessively.

The following sterilizing methods are forbidden:

- Put handpiece into any liquid for boiling.
- Dip the handpiece in disinfectors such as iodine, alcohol and glutaraldehyde.
- Put the handpiece into an oven or a microwave oven for baking.
- Put the handpiece into ultrasonic cleaner to clean.

## 7. Storage, maintenance and transportation

### 7.1 Storage, maintenance

- The equipment should be handled carefully and lightly. Be sure that it is far from vibration, and is installed or kept in a cool, dry and ventilated place.

- Don't store the machine together with articles that are combustible, poisonous, caustic or explosive.
- This equipment should be stored in a room where the relative humidity is 10% to 93%, atmospheric pressure is 70kPa to 106kPa, and the temperature is -20°C to +40°C.

## 7.2 Transportation

- Excessive impact and shaking should be prevented in transportation. Lay it down carefully and lightly and don't invert it.
- Don't put it together with dangerous goods during transportation.
- Avoid solarization and getting the product wet due to rain or snow during transportation.

## 8. Environmental protection

There are no harmful factors in our product. You can deal with it based on the local law.

## 9. After sales service

We offer one year's free repair to the equipment according to the warranty card. The repair of the equipment should be carried out by our professional technician. We are not responsible for any irretrievable damage caused by a non-professional person.

## 10. Symbol instruction

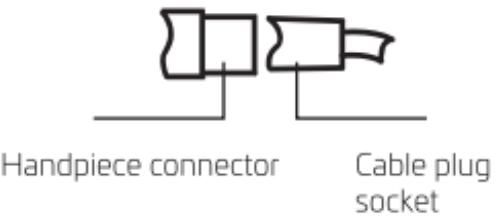
	Consult the accompanying documents		Drip-proof
	Date of manufacturer		Used indoor only
	Manufacturer		Can be autoclaved
	According to the type of protection against electric shock: CLASS II EQUIPMENT		pliance compliance WEEE directive
	According to the degree of protection against electric shock: Type B applied part		Humidity limitation 10%~93%
	Courant alternatif		Atmospheric pressure for storage 70kPa~106kPa
	Temperature limitation -20°C~+40°C		

## 11. Statement

All rights of modifying the product are reserved to the manufacturer without further notice. The pictures are only for reference. The final interpretation rights belong to the manufatruer. The industrial design, inner structure, etc, have been claimed by several patents by manufacturer. Any copy or fake product must take legal responsibilities.

### ⚠ Note

Before installing the handpiece, make sure that the handpiece connector and the cable plug socket are completely dry.

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**MedNet EC-REP GmbH**
**Guilin Refine Medical Instrument Co., Ltd**