ULTRASONIC CLEANER

USC-2.5L

Nettoyeur à ultrasons
Ultrasonic Cleaner
Limpiador ultrasónico
Banho de ultrassons
Vasca ad ultrasuoni
Ultraschallreiniger
Ultrasoonreiniger

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ULTRASONIC CLEANER USC-2.5L

en

SAVE THESE INSTRUCTIONS IN A SAFE PLACE

1. FEATURES

1	Lengthened tank	Suitable for long dental equipment and tube
2	Stronger transducer	To get better cleaning effect
3	5 cycle digital timer	Choose different time to clean
4	Heater	Safety and reliable with temperature (65 °C) and time (45 min) controller
5	Circuit protector	Protect the circuit to lengthen unit life
6	Radiator Fan	To protect every parts in good condition after working a long time
7	Damp-proof PCB	Good to be used in lab
8	Three separable wires	Safety and convenient
9	Industry IC	To be steady in unstable voltage condition
10	Strong plastic housing	Better water-proof and drop-proof ability than metal

2. INTRODUCTION

No special solutions required, only use plain tap water

Ultrasonic cleaning technology uses water as media, with high frequency vibration releasing billions of small bubbles. These bubbles break with power, which removes dirt from cleaning articles safely and effectively.

Definition of Main Features

With 5 circle time setting and heating device, with a bigger capacity (2500 ml), and stronger cleaning power (160 W). Industrial spare parts being adopted, allows its working in long time frame, and in dampness environment. In this case, the use range has been expanded.

Read the user's guide

Ultrasonic cleaners are very easy to use. Although to get the best from your new purchase, we recommend you read this user guide before getting down to any serious cleaning. It also contains important safety information as well as practical hints.

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

3. SAFETY INSTRUCTIONS

⚠ DANGER

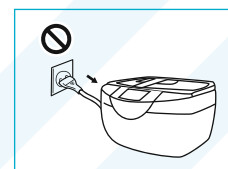
3.1. To reduce the risk of electrocution

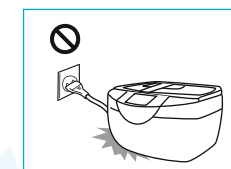
Unplug this unit immediately after using.

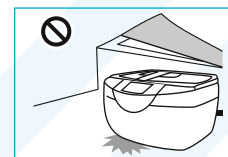Do not immerse the unit into water or other liquid.

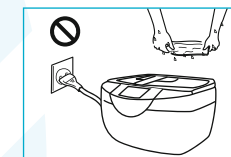
Keep the unit in a safe place to avoid falling into water or other liquid.

Do not use while bathing.

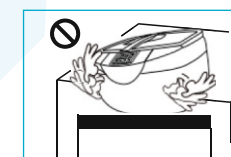Do not reach for the unit that has fallen into water. Unplug it immediately.


Do not place or store product where it can be pulled into a tub or sink.

- Do not place or drop into water or other liquid.
- To avoid damage caused by high temperature, please shut down the cleaner for no less than 3 minutes and change the used water after three 480s working circles.
- The product may be off or can't be switch off when it is subjected to electromagnetic interference like Fast Transients. If this occurs please follow the instructions described in the manual to set the function again for operation.
- Don't use the product if there isn't water in the cleaning tank.

⚠ WARNING

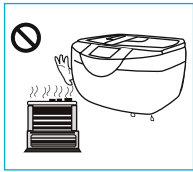3.2. To reduce the risk of burn or injury to persons

Keep the unit and the cord away from heated surfaces, and place it on a dry and flat surface.

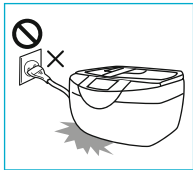
The unit should never be left unattended when plugged in.

Do not disassemble the unit except for the authorized technician.


Do not fill the tank with abrasives or corrosive chemicals.

Unplug the unit before filling. Do not exceed the «MAX» mark.

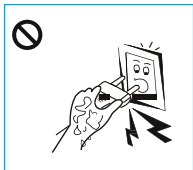
Do not immerse the unit into water to avoid electric shock.

Do not touch the socket with wet hands to avoid electric shock.

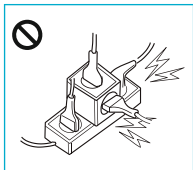Do not use this unit when sleeping or drowsy, and do not use outdoors or operate where aerosol (spray) products are being used or where oxygen is being administered.


Use the unit properly as per its instruction manual. Do not use attachments not recommended by the manufacturer.

Never operate this unit if cord or plug is damaged. If it is not working properly, damaged or has been dropped into water, return the damaged unit to a service centre for examination and repair.

	<p>Never drop or insert any object into any opening or hose. Never block the air openings of the unit or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep the air openings free of lint, hair, similar, ect.</p>
	<p>Environment friendly disposal. You can help protect the environment! Please remember to respect the local regulations: hand in the non-working electrical equipment to an appropriate waste disposal centre.</p>

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
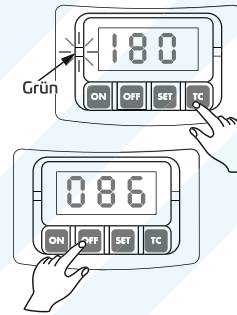
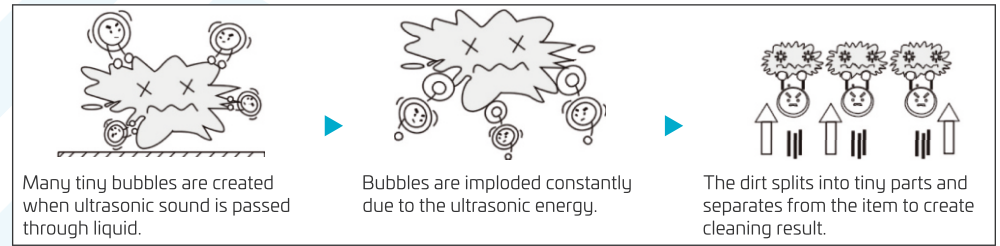
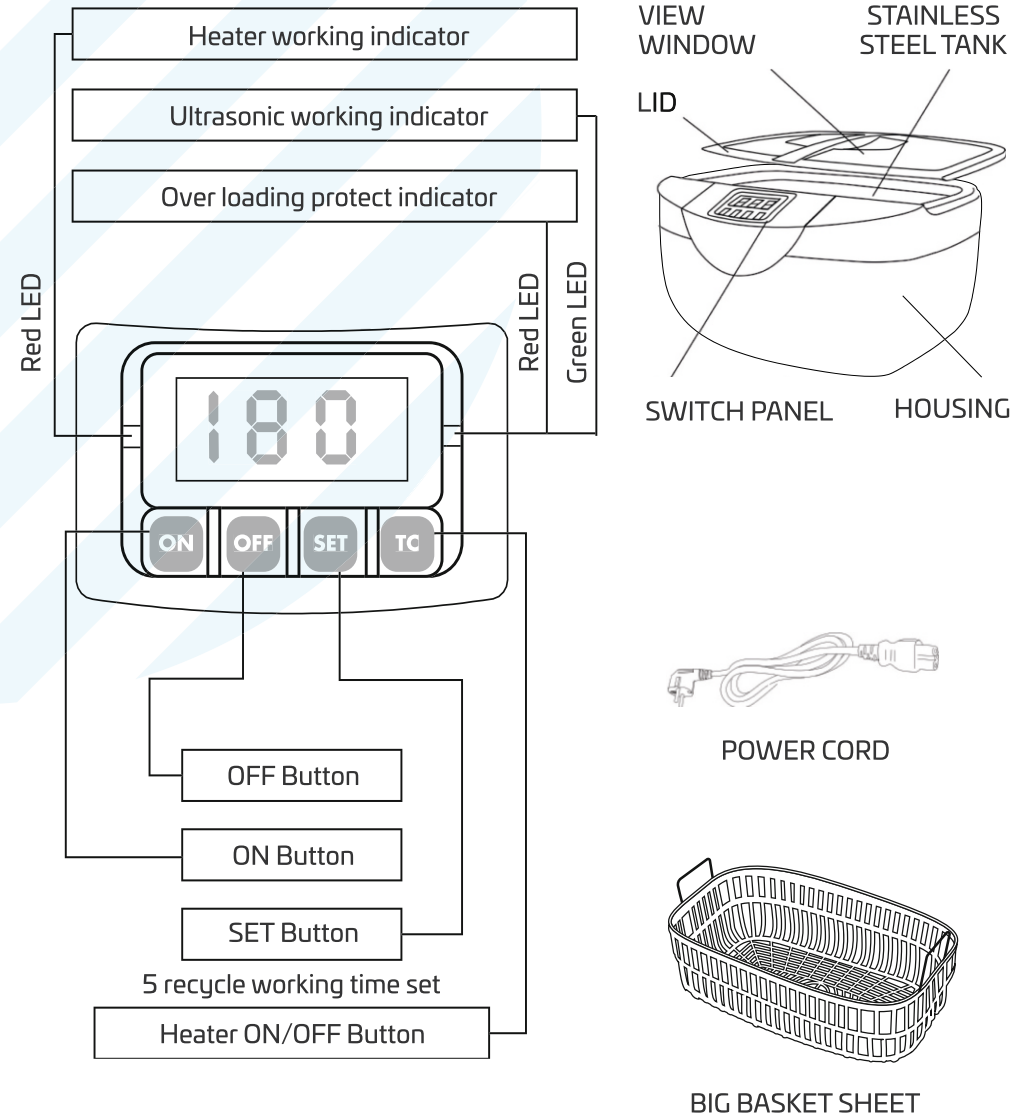
4. FONCTIONS AND ACCESSORIES

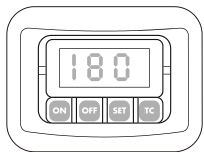
4.1. Regarding 5 digital time choosing

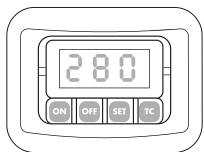
It will automatically display 180s, it is the most common cleaning time. Pressing ON starts cleaning. The time displayed is the time left. When the time "000" is displayed, the unit will automatically stop.

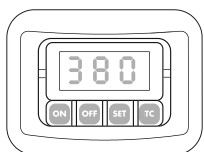
Every time you press "SET", it will change and display another time selection. It is usually used as the first step cleaning of the instrument. The cleaning will take longer if the filth stayed for a long time on the item.

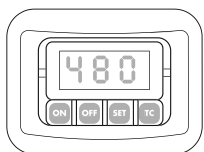Usually used for cleaning several products together. Choose this to shorten the cleaning process.

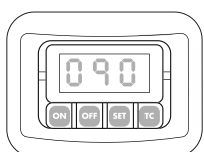
This time display is usually used to clean greasy metal parts, this time choosing, using the heater function. This function can soften the oils to increase the cleaning result.

This step is usually taken after finishing the above steps, but the water in tank is dirty, replacing with clean water and then cleaning another 90s to keep clean of the product's surface.

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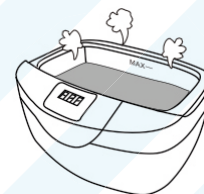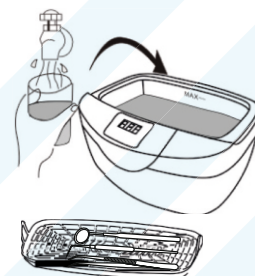
5.1. General Cleaning


Only using tap water

Put the item to be cleaned on the mesh screening, then fill the water. Ensure the item is immersed in the water; 1 the water level should above the “MIN-” marking, but does not exceed the “MAX” marking. Put in the plug and press the “ON” button.

Press “SET” button, choose one circle from 280/380/480S, then press “ON” button, the unit start to work.

Note: While cleaning, the filth appears on the instrument and looks like «smoke». Meanwhile, the clean water will get dirty. The item is clean if there is no «smoke», no need to clean it one more time. (Usually in 180s is sufficient). If the water is dirty, change it with clean water. Continue with cleaning another item.

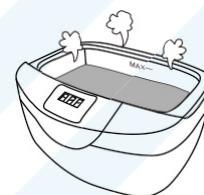5.2. TURN ON THE HEATER FUNCTION

Put the item on the mesh screening, and then put the mesh screening into the stainless steel tank. Keep the item immersed in the water, ensure the water is above “MIN-” marking but does not exceed the “MAX-” marking. Press the “TC” button, the heater is working, around 40 minutes later, the temperature of the water is about 65°C, the heater will auto shut off.

To reduce the time of heating, put in some warm water, but don't put in boiled water.

Press “SET” button, choose one cycle from 280/380/480S, then press “ON” button, the unit start to work.

Note: Using 65°C warm water, can enhance cleaning.

5.3. Put a cleaning product to enhance cleaning

Keep the item is immersed in the water, with the cleaning product. “MIN” marking but does not exceed the “MAX” marking.

Note: The liquid soap is neutral, it is a “soft” chemical liquid.

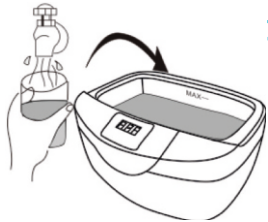
Press “SET” button, choose one circle from 280/380/480S, then press “ON” button, the unit starts to work.

! If using a cleaning product, you need to wash the item again with clean water, or using clean water go on cleaning 90S, in order to clear away the cleaning product.

5.4. How to use mesh screening

- Using mesh screening can avoid the attrition between the item and stainless steel tank, it also can reduce the attrition between items.
- When put the item and mesh screening into the tank, the unit also can clean the mesh screening, therefore, the mesh screening can absorb 15% ultrasonic energy. (Plastic basket can absorb 20% ultrasonic energy).


6. OPERATING INSTRUCTIONS

- 1 **Open the lid**, put instruments on the shift in line then put the shift in the tank.

⚠ *To avoid the scratch and friction of the products, do not overlap them.*

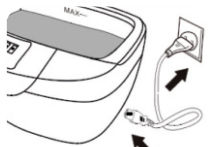
Add the water in the stainless steel tank, the water should over the surface of the product but within the range of MIN mark and MAX mark.

- 2 Insert one side of the detachable plug and cord into the body of the machine, the other to the socket. (See picture).


180s will be displayed when the power is on. Press "ON", it begins cleaning. 180s is the most common working cycle. During cleaning, the time displayed on the screen is the time left.

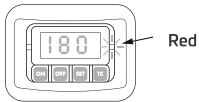By pressing "SET", you can choose 5 optional working cycle: 90S / 180S / 280S / 380S / 480S

- 3 Press "ON" button to start cleaning while the right green indication light is on.

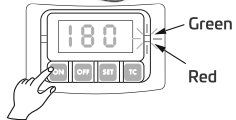⚠ *During cleaning, you can hear the "BUZZ" voice; it means the unit is working.*



When the timer counts down "000", the unit will automatically stop. If required you can stop / pause the cleaning process at any time by pressing the "OFF" button, pressing the "ON" button again will resume the cleaning and the countdown will continue. When cleaning has finished, you can repeat the process by pressing "ON" button again.


⚠ *This machine has a circuit protecting it when over heat. When the right red indication light is on, it means this circuit works. If pressing "ON", the time left will be displayed but there is no ultrasonic wave. The machine needs to stop working for 15 minutes, then pressing "ON" and the right green light is on, the machine can work normally.*

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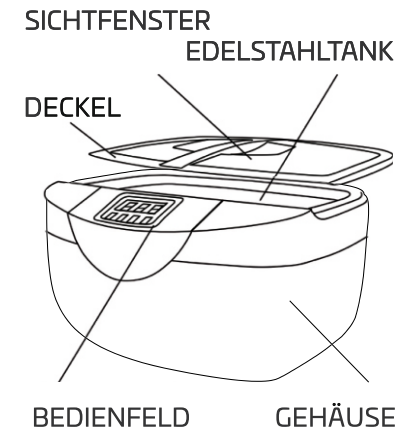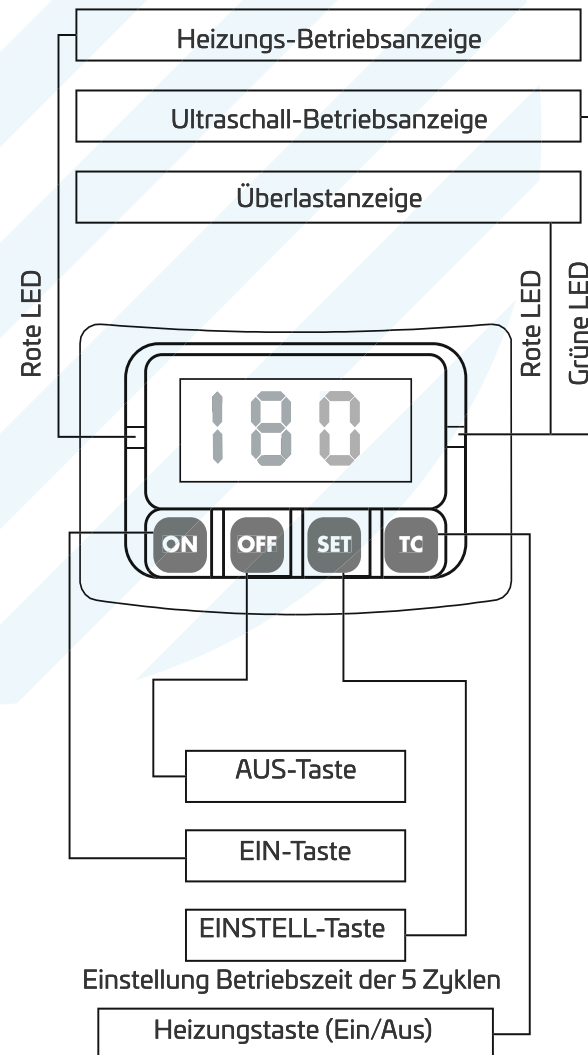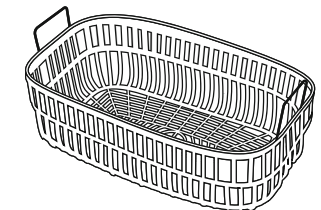
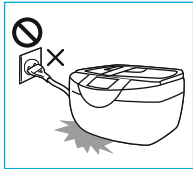
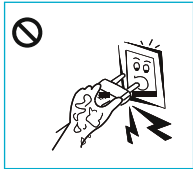
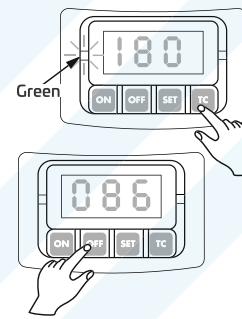
4 Using the HEATER function.

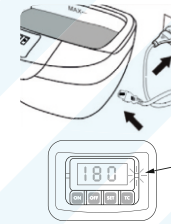Pressing the TC button starts the built in heater. The red LED on the left of the control panel will illuminate. The water will heat to a maximum of 65°C. Please refer to the 5 point of four different ways of cleaning for details. If required, you can stop / pause the cleaning process at any time by pressing the "OFF" button, pressing "ON" again will resume the cleaning and the countdown will continue. The program can be reset at any time by pressing the "SET" button.


5 When cleaning is finished, unplug it and open the lid, then take out the item. Put away the water from stainless steel tank and wipe it up.

7. USER MAINTENANCE

1 This machine has taken several protective measures to the circuit, but if working without water (over 20 seconds) will short the life of the machine.

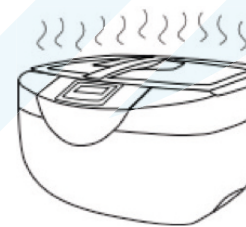⚠️ *Damage will be occurred if works without water.*

2 This machine has a protective circuit when it overheats. It is recommended 15 minutes break, between 3 working continuous cycles.

3 Unplug and empty the unit after using.

⚠️ *Do not keep water in the tank for a long time.*

4 Wipe the tank and unit with dry towel.

⚠ Do not wash the complete unit with water to avoid the risk of shock.

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