EC REP

**Lotus NL B.V.**

Koningin Julianaplein 10, 1e Verd, 2595AA, The Hague, Netherlands.  
E-mail: peter@lotusnl.com

CE  
0 1 9 7

**Foshan COXO Medical Instrument Co., Ltd**

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# HANDPIECE HP-40

Please read this user's guide carefully and file for future reference.

## Preface

Thank you for purchasing our dental straight handpiece. In order to use this equipment safely and effectively, please read these instructions carefully before use, and file them for future reference.

For regulatory reasons, we are committed to the optimization of our products. This may cause some parts to change. If the product differs in any way from the description in the manual, please contact your supplier or our after-sales center for details.

## 1. Description

Low-speed straight handpiece, guaranteeing the stable and reliable performance required of all equipment used for general dental treatment and daily dentistry work.

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## 2. Technical specifications

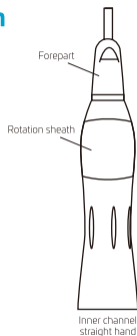
Relative humidity: 10% to 80%, non-condensing  
 Noise: ≤70 dB  
 Permissible max. speed: 40,000 rpm  
 Burs:  $\Phi$  2.35 Handpiece bur

### WARNING

Please read this operations manual carefully before use, and file for future reference.  
 This product is specialized for stomatology; do not use it in any other way.  
 This product is intended for registered dental/medical professionals only.  
 Remember to check the equipment before each use. If you notice anything unusual, stop using it immediately, and contact your supplier.  
 Before operating this low-speed handpiece, gently pull on the head of the straight handpiece to make sure that it is firmly and safely connected.  
 Do not operate without inserting a bur; otherwise the head or motor may be damaged.  
 Do not immerse the handpiece in any chemical solvent or solution, or attempt dry-heat disinfection.  
 Instead, sterilize by autoclave at 134°C for at least 18 min.

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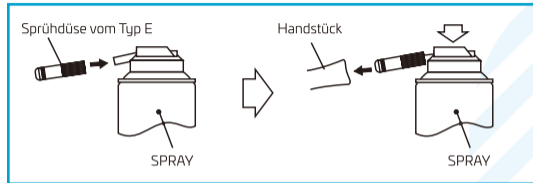
## 4. Mounting the bur on the handpiece

When you need to insert or remove a bur, move the bur lock ring to the right until it "clicks", then open the chuck, insert or remove the bur, and move the lock ring to the left it "clicks" again. The chuck should now be locked.

The part under the bur should rotate smoothly.

Hand-tighten again if necessary.

The straight head can now be used.

### **Tips for safe use and maximum service life**

Never use bent, off-centered, or damaged burs.

Excessive loads generated by defective burs rotating at 40,000 rpm will quickly damage the ball bearings.

Always insert a bur all the way into the chuck.

This will prevent bur walk-out and/or damage to the ball bearings.

Never operate without a bur or bur blank securely in place, or with the bur lock ring in an incorrect position.

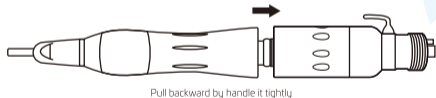
## 5. Handpiece and motor connections

### 5.1. Assembly:

Hold the motor tightly to insert the handpiece into the air motor.  
Make sure the groove is aligned to insert the handpiece into the air motor.

### 5.2. Disassembly:

Hold the motor tightly and pull the handpiece out.

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## 6. Clean-head system

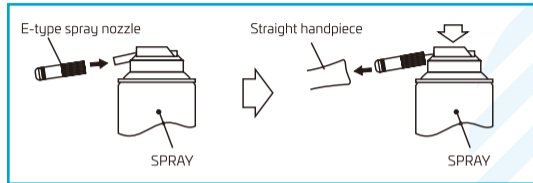
- 1) Scrub any impurities and debris from the handpiece.
- 2) Make sure the handpiece is not on, and wipe it dry. If impurities remain in the hole, remove the cartridge and use a brush to clean it.

Only use clean water for all maintenance of the clean-head system.

## 7. Lubrication

Apply lubricant spray after each use and before autoclaving.

- 1 Push the E-type spray nozzle attachment onto the spray nozzle until it sits firmly.
- 2 Shake the can 3-4 times to ensure the lubricant and propellant are well mixed.
- 3 Insert the E-type spray nozzle into the rear of the handpiece and spray for approximately 2-3 seconds until oil comes out of the handpiece head.

### ⚠ CAUTION

Hold the handpiece firmly to prevent it from slipping out due to the pressure of the lubricant spray.  
Continue to spray lubricant until it comes out of the handpiece head (approx. 2 seconds).  
Keep the lubricant spray can upright.

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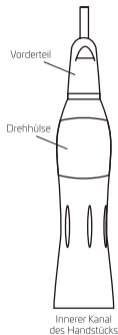
## 8. Cleaning and sterilization

### 8.1. Cleaning:

Wipe the handpiece sheath clean with a soft tissue or cotton wool dipped in alcohol. Never clean the handpiece in boiling water or in chemicals, with wire brushes, or in ultrasonic cleaners.

### 8.2. Sterilization:

Use of a steam autoclave is recommended.

Autoclave sterilization is mandatory after each patient, following the procedure below.

- 1 Clean the handpiece sheath as described above.
- 2 Lubricate the handpiece with lubricant spray (ref. lubrication part).
- 3 Place the handpiece into the autoclaving pouch and seal it in accordance with the instructions on the pouch.
- 4 Autoclavable at 134°C.

## 9. Troubleshooting

Problem	Possible cause	Solution
The body of the straight head and contra angle rotate when the motor is running.	The o-ring on the nose of the motor is worn-out.	Change the o-ring.
The handpiece fails to rotate.	The contra angle is chucking with the bur, but the latch plate is not fixed in the proper position. Contaminants can enter the handpiece.	Clean and lubricate the handpiece, and adjust the latch plate manually.

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## 10. Warranty

CFPM grants the user a 12-month warranty for its complete product range. Maintenance beyond the term of the warranty will be at the customer's expense. CFPM will not be responsible for damage or injury resulting from:

Excessive use.

Improper manipulation of the product or modifications to the product carried out by persons not authorized by CFPM.

Failure to follow the instructions to install, operate, and maintain the handpiece.

Chemical, electrical, or electrolytic damage due to improper autoclaving and storage.

Improper working pressure.

The technical specifications and dimensions in these instructions are given only as a guide. They may not be subject to any claims.

## 11. Transportation, operating, and storage conditions

Operating temperatures	+5 to +40°C
Operating humidity	≤80% RH
Storage temperatures	-10 to +55°C
Storage humidity	≤93% RH

## 12. Standard symbols

The following standard symbols appear on the instrument label:

Attention: consult supporting documents

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