fr: Poudre prophylactique
 en: Prophylaxis powder
 es: Polvo profiláctico
 pt: Pó profilático
 it: Polvere profilattica
 de: Prophylaxepulver
 nl: Profylaxe poeder

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en The prophylaxis powder is used with the dental air polishing and microblaster devices to remove the dental plaque, tartar, calculus, etc.

1. Main composition

Sodium bicarbonate (< 65 µm), hydrophobic silica, sodium saccharinate and food flavour.

2. Operation method

- Before opening the cap, shake the bottle up and down with hands for 3-5 times.
- Open the cap and pour the appropriate amount of powder into the powder tank of the device according to the patient's oral condition, and then tighten the cap.
- Activate the device and align the nozzle with patient's tooth surface to start scaling.

3. Contra-indication

- Under no circumstances should this powder be applied to patients with chronic bronchitis or asthma.
- This powder should not be used to treat patients with a low-salt diet.
- The essence contained in the powder may trigger an allergic reaction under certain conditions. For patients who are known to be allergic to essence, please stop using this powder.
- Do not align nozzles directly to fillers, crowns, and bridges as this may damage these restorations.
- It is forbidden to apply this powder to subgingival part.

4. Storage and shelf life

- Powder should not be stored near acidic liquids and heat sources.
- Store away from damp and light, at a temperature between -10 °C and 40 °C.
- Period of validity: Three (3) years from the date of production.
- Do not use after expiration date.

5. Treatment advice

- Before treatment, apply Vaseline to the patient's lip.
- Patients and operators must wear goggles during treatment.
- The user must wear a mask during treatment.
- Use the high-speed emptying device on the dental unit to absorb the. Use the rapid suction device of the dental unit to absorb sprayed powder during treatment.

6. Precautions

- After treatment, do not put the remaining powder back to the powder bottle.
- After opening the powder bottle, cover it with the cap and tighten it in time to protect the powder from moisture.
- When the air polishing device is not used, please clean the powder tank to prevent the powder being agglomerated and block the device.

7. Transport

- Avoid mixing with acidic substances and dangerous goods during transport.
- During transport, avoid exposure to sun, rain and snow.

8. Environment protection

This product does not contain harmful ingredients and can be disposed of and destroyed in accordance with local regulations.

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EC REP MedNet EC-Rep GmbH
CE Guilin Woodpecker Medical Instrument Co., Ltd.