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Attention, consulter les documents joints

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Intended Use: Dental Treatment

CAUTION

- Read this user's guide before use to fully understand the product functions and file for future reference.
- When operating the Handpiece always consider the safety of the patient.
- Check for vibration, noise and overheating outside the patient's oral cavity before use. If any abnormalities are found, stop using the Handpiece immediately and return it for reparation.
- Should the Handpiece function abnormally, cease operation immediately and contact the supplier.
- Depressing the push-button while the handpiece is rotating will result in OVERHEATING of the handpiece head. Special caution must be exercised during use to keep soft tissue AWAY from the push-button of the handpiece. Contact with the cheek may cause pressure on the push button and may cause a burn.
- Do not attempt to disassemble the handpiece or tamper with the mechanism.
- Do not allow any impact on to the Handpiece. Do not drop the Handpiece.
- Do not connect or disconnect the Handpiece or bur until the drive motor has completely stopped.

CAUTION

- Do not use bent, deformed or damaged bur. If these burs are used, they could bend or suddenly break during rotation. Bur may become difficult to remove because latch groove is worn or it has expanded.
- Always keep burs shanks clean. Dirt or debris in a chuck could cause poor bur concentricity or low chuck retention force.
- Do not exceed the drive motor speed recommended by the bur manufacturers.
- Do not apply excess pressure to the bur as it may break or bend or become difficult to remove.
- Before use always confirm that the bur is correctly seated and locked.
- Always use with adequate water supply and coolant air or damage to the tooth structure and overheating may occur.
- If a foreign material enters the gears or internal mechanism of the handpiece, it may result in burn injuries.
- Use eye protectors, dust mask, for safety and health during treatment.
- Perform periodical maintenance checks.
- If the handpiece has not used for a long period, check for noise, vibration, and overheating before use.
- Do not wipe nor immerse the handpiece acidic water or acidic solutions.
- User is responsible for operation, maintenance and safety.
- **Do not use the handpiece if the bur is not correctly mounted, otherwise the chuck might be thrown by the reactive force and the bearing support might be damaged.**

1. Specification

Model	RED L-200
Max.Rotation Speed	200 000 rpm
Max.Rotation Speed	1:5
Bur	ISO1797-1 Type 3 Ø 1,60 mm Fraise FG
Optic	Glass Rod
Bur applied Length	10,9 mm
Max Bur Length	25 mm
Max Bur diameter	1,60 mm
Water Spray Type	Quadruple Spray
Use Environment	Temperature: 5-40°C (No Condensation), Humidity: 20-80%
Transportation and Store Environment	Temperature: -10-55°C Humidity: 20-80%, Atmospheric Pressure: 86-106 kPa

2. Connecting/ Disconnecting Handpiece and Motor

2.1 Connecting

- 1) Align the handpiece and motor alignment pin.
- 2) Then push the handpiece directly into the motor in alignment with the pin.

2.2 Disconnecting

Hold the motor and the handpiece, and pull it out.

CAUTION

Connect ONLY E type motor (ISO3964) and do not exceed maximum motor speed such as shown on specifications.

3. Check before treatment

Follow the checks below, before use. If any abnormalities are found, stop using the handpiece immediately and contact the authorized dealer.

- Check that the Head Cap is firmly tightened.
- Insert the Bur. (Do not exceed maximum rotation speed for any item.)
- Check that there is no abnormal rotation, vibration, noise and overheating.
- If there are no immediate apparent abnormalities, continue to rotate the handpiece for about 1 minute. Then check that there is no abnormal heating on the outer surface.
- Operate only after checking for the abnormalities above.

CAUTION

Do not use the handpiece if the bur not be mounted, otherwise the chuck might be thrown by the reactive force and the support might be damaged.

4. Mounting / Removing the Bur

4.1 Mounting the Bur

- 1) Insert and lightly turn the bur until it engages with the latch mechanism. (Fig.4.1)
- 2) Push the bur further inwards until it clicks.

- ⚠ CAUTION**
- After the Bur is locked, lightly pull out the Bur to make sure the Bur is locked.
 - If the Bur is not correctly secured it may release during operation and could cause injury.
 - **Do not use the handpiece if the bur is not correctly mounted, otherwise the chuck might be thrown by the reactive force and the support might be damaged.**

4.2 Removing the Bur (Fig. 4.2)

- 1) Push the push button with your thumb.
- 2) Pull out the bur.

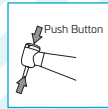
Fig. 4.1

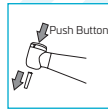
Fig. 4.2

5. Maintenance Method

After use, be sure to perform the maintenance below.

This symbol on your product means that the product can be autoclavable up to Max. 134°C.

5.1 Preparation

Remove Contra Angle from the motor and detach the bur from the chuck.

5.2 Cleaning

- Carefully clean all surfaces with a cloth impregnated with Spray.
- Wipe clean the Cellular Glass Optic Rod entry point and exit point with an alcohol-immersed cotton swab. Remove all debris and oil.(Fig.5.1).

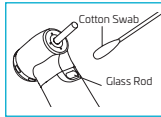
Fig. 5.1

CAUTION

Do not use a pointed tool or sharp edged to clean the Cellular Glass Optic Rod. They may damage the mirror finish and reduce the light transmission efficiency. If the glass rod is damaged, return the device for reparation.

5.3 Disinfection

Wipe the external surface of handpiece carefully with cleaning or disinfectant solution.

5.4 Lubrication

To prevent mist spray from escaping into the environment, use absorbent cloth to absorb excess lubrication.

- Push or screw nozzle. Shake the spray can 3-4 times.
- Insert the spray nozzle in the rear of the handpiece and spray for approximately 2-3 seconds until the oil comes out of the handpiece head / nose.

Automated:

Clean and lubricate handpieces be sure to use a right adaptor for each handpiece.

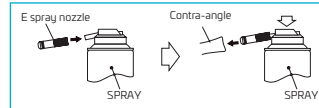
Fig. 5.2

CAUTION

- Be sure to hold the handpiece firmly to prevent the handpiece from slipping out from the spray pressure when lubricating.
- Keep the can upright.
- Use a lubricant suitable for contra-angles.

5.5 Sterilization

Autoclave sterilization is recommended.

Sterilization is required first time you use and after each patient as noted below.

Autoclave procedure:

- Insert into an autoclave pouch. Seal the pouch.
- Autoclave for 18 minutes at 134°C.

CAUTION

- Always place the handpiece in the center or upper shelf of the autoclave, as the local temperature at the bottom of chamber could rise beyond the set value.
- Do not autoclave the handpiece even when it is bagged, together with other instruments, from which chemicals are not sufficiently cleaned . This is to avoid discoloration and/or damage to the coating of the handpiece.
- Do not heat or cool the handpiece quickly. Rapid change in temperature could break the glass rod and give abnormal strain to other metals.
- The handpiece is kept in suitable atmospheric pressure, temperature, humidity, ventilation, and sunlight. Moreover, air does not include dust, salt and sulfur.
- We do not recommend you to sterilize by plasma sterilization or EOG sterilization.

6. Warranty

The manufacturer grants the user a 12 months guarantee for its complete product range, except ball bearings (3 months guarantee) from the date of invoice issued. Maintenance over the term of guarantee will be at the customer's charge.

The manufacturer will not be responsible for damage or injury resulting from:

- Excessive use
- Improper manipulation of the product, or modification to product carried out by persons not authorized by the manufacturer.
- Failure to follow the instructions for installation, operation and maintenance of the handpiece.
- Damage of chemical, electrical or electrolysis due to improper autoclaving and storing.
- Improper working pressure.

The technical specifications and dimension in these instructions are given only as a guide. They may not be subject of any claim.

7. Standard Symbols

On the instrument label appear standard symbols as follows:

CE Marking

Sterilizable in an autoclave up to max.134°C

Moisture limitation

Can be used with a washer-disinfector for thermal disinfection

Attention, consult accompanying documents

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